




NEW!


Medicare Guidelines for Reimbursement National Coverage Determination (NCD) for RAD and HMV



NCD for RAD Devices

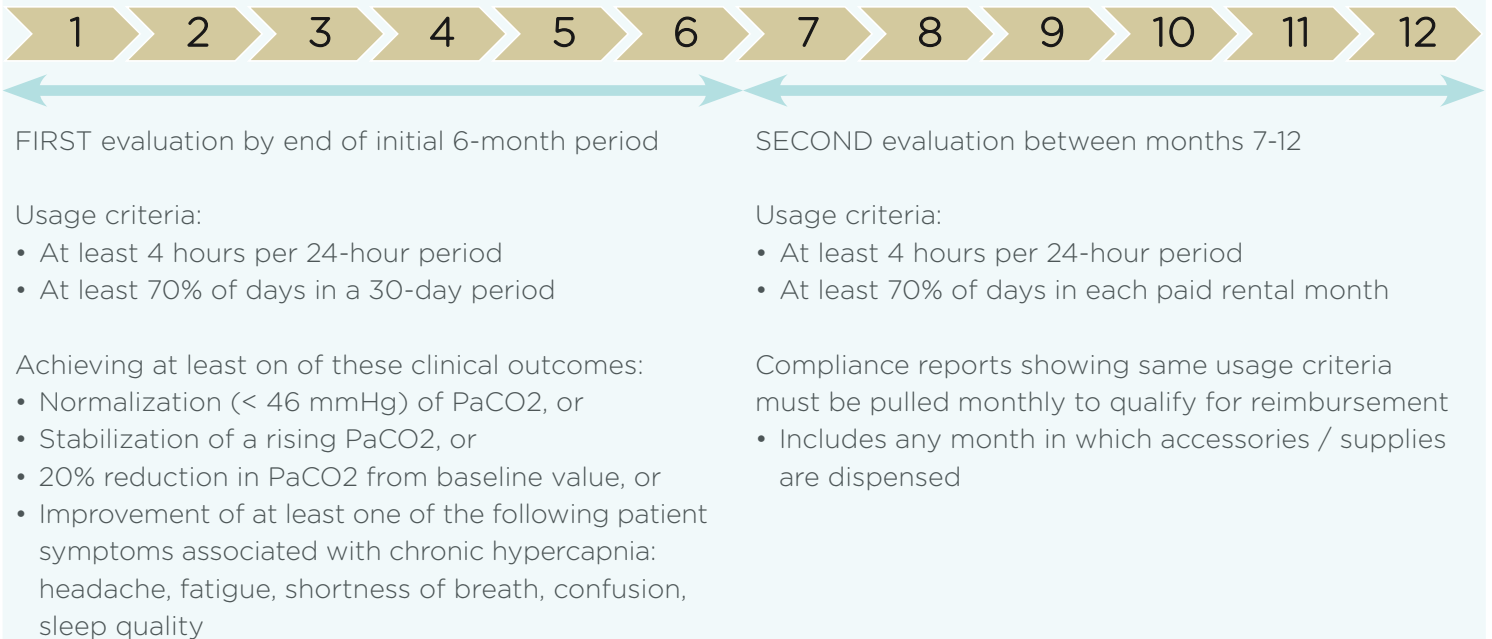

NCD for HMV

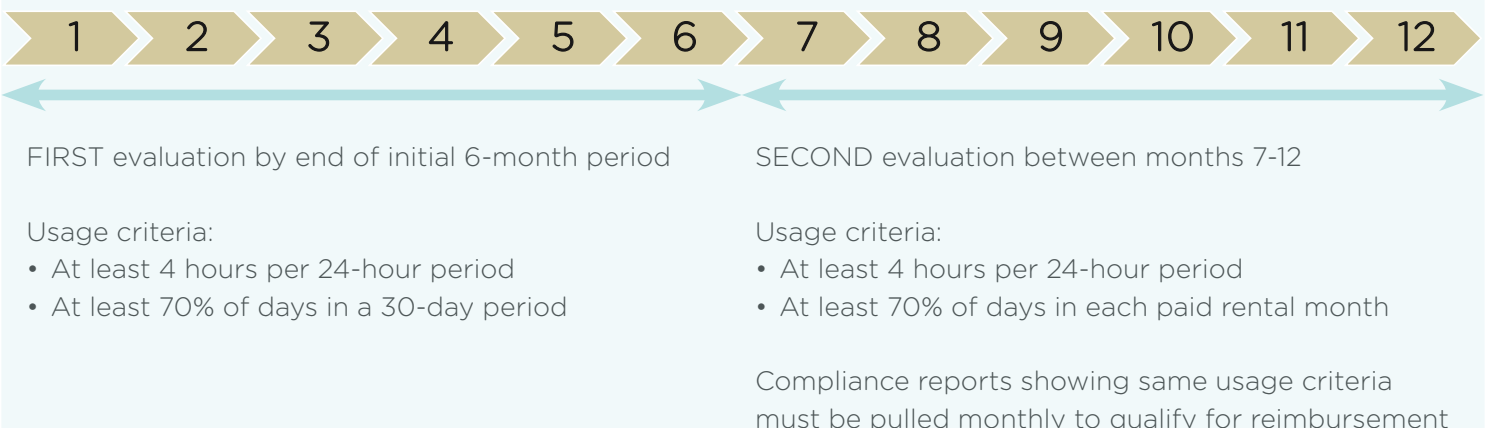

Enhanced EveryWare Reporting

DESIGNED TO COMPLY WITH NEW NCD GUIDELINES

One-Page Report

- Purpose built report aligns with new NCD Compliance Guidelines
- Available via EveryWare and API
- Specify start and end date:
 - Specify start and end date
 - 180 days from set-up date
 - 30 days from end date

One-Click Patient List

- Expanded functionality of the exportable patient list
- Now includes: Number of days supporting NCD Compliance, date and time of most recent modem connection, current compliance threshold, and patient's date of birth

Patient Compliance at a Glance

Patient *****
Report for 29-Aug-2025 - 27-Sep-2025
Created On 28-Oct-2025 07:19:47 PM



Patient Information
First Name ***** Last Name ***** Gender *****
Patient ID ***** Date of Birth ***** Age *****

Ventilator Information
● N360179 (Vivo 45 LS US)

Compliance
Compliance % (Usage ≥ 4 hours/day) 83 %
Days with Compliance 25 / 30 Days
Day with Usage % 83 %
Days with Usage 25 / 30 Days
Total Usage Hours 277:01 Hours
Average Usage Hours (Days Used) 11:05 (277:01/25) Hours
Average Usage Hours (Total Days) 9:14 (277:01/30) Hours
Total Mode Usage Hours PSV(TgV)-AE 277:01 Hours


Therapy Statistics

	Minimum	Median	90%	Maximum
Ppeak (cmH2O)	3.50	17.00	20.00	33.00
PEEP (cmH2O)	0.25	12.00	14.00	17.00
Total Leakage (L/min)	1.00	106.00	121.00	121.00
Tidal Volume (ml)	25.00	400.00	550.00	3025.00
Minute Volume (L)	0.25	4.50	9.00	30.25
Breath Rate (bpm)	10.00	12.00	20.00	39.00

Respiratory Indices

AHI	AI	HI	Spontaneous breaths (%)
40.80	0.00	0.00	54.04

	Minimum	Median	90%	Maximum
Insp. Time (s)	0.25	1.00	1.50	6.25
I:E Ratio	1:5.0	1:3.0	1:1.8	1:0.6

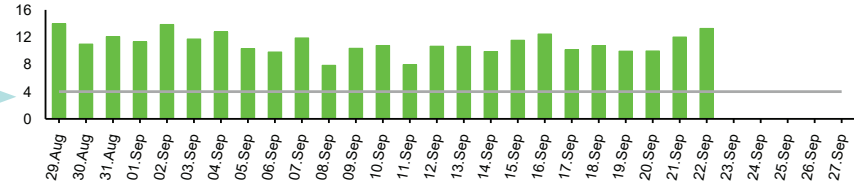Hours in Use (h)

Compliance %
≥4 hours within
30 days

Highlights
compliance based
on the best 30-day
period from the
user-specified
date range

Usage Graphs show
utilization of best
30 days versus
4-hour threshold

To capture more compliance information about a patient, simply choose the standard 4-page report.



Breas and the Breas logo are trademarks of Breas Medical AB. © 2025 Breas Medical, Inc. All rights reserved. Breas Medical reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your Breas representative for the most current information. BOS-MAR-MAR-008182 Rev 0

Breas Medical Inc · 16 Esquire Rd · N. Billerica · MA 01862 · USA · Phone 855-436-8724 · www.breas.us